

Hinged Support Rail and Grab Rail

Introduction

This installation guide has been produced for the Inta hinged support and straight grab rails which are available in a range of different lengths and white or blue colour.

These instructions cover the installation and maintenance.

Please read the enclosed instructions before commencing the installation of this product, please note;

We recommend that the installation of any Inta product is carried out by an approved installer.

Product Range

Straight rail lengths available 300mm, 450mm and 600mm.

Hinged support rail with toilet roll holder.

Preparation

Before commencing please ensure the wall and surface onto which the grab rail(s) are to be fixed is structurally suitable and capable of taking the required loading.

When positioning the grab rail(s) please ensure the optimum position for the convenience and safety of the user which should be approximately 80 cm from the floor.

The straight rail can be installed horizontally, vertically or inclined.

The hinged support rail must be installed horizontally.

Screws and basic wall plugs are supplied, suitable wall plugs must be selected to suit the type of wall whether studded, soft building block or brick.

The maximum diameter of hole in the fixing plate is 6mm.

Ensure there are no supply pipes or electrical cables where you intend to drill.

Installation - Straight Grab Rail


The straight grab rails are supplied assembled with the concealing plates fitted.

Before drilling into the wall, physically hold the grab rail against the wall and mark the position for the holes to be drilled.

The 3 fixings holes in each fixing plate should be used to fix and support the grab rail.

Ensure that suitable fixing screws are tightened securely.

When fixed to the wall, push the concealing plates towards the wall until they click into position.

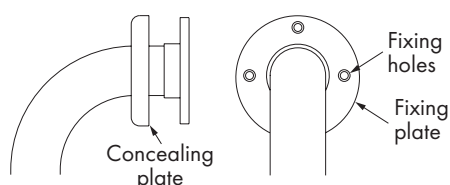

Installation - Hinged Grab Rail

Before drilling into the wall, physically hold the grab rail against the wall and mark the position for the holes to be drilled.

The 4 fixings holes in the wall plate should be used to fix and support the grab rail.

Ensure that the supplied fixing screws are tightened securely and then fit the covers to the screw heads.

Maximum Load


The maximum load to be applied to the straight or hinged grab rail is 120 kg, which must never be exceeded.

The grab rails are not designed to support the total weight of the user, they are intended for assistance **ONLY**.


DO NOT use the grab rail as a step, to sit on or hang heavy articles from it.

Dimensions

Straight Grab Rail

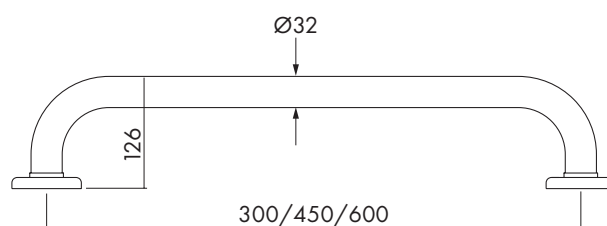

Hinged Support Rail

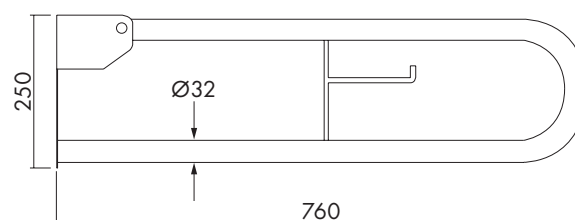

Aftercare

The grab rails are supplied with a durable powder coated finish.

An occasional wipe with a mild washing-up liquid on a soft damp cloth followed by a thorough rinsing is all that is required.

For additional protection against bacteria a disinfection agent may be used, the manufacturer's recommendations must be followed.

Do not use an abrasive or chemical household cleaner as this may cause damage.

Please leave these instructions for the user